# SIMPLEE®

All you need. No more. But no less.

## RETRACTION PASTE

### INSTRUCTIONS FOR USE

**STORE THIS PRODUCT IN A COOL, DRY PLACE**  
Refrigeration will prolong shelf life, but avoid freezing. Bring to room temperature before use.

**PRODUCT DESCRIPTION**  
Simplee Retraction Paste is a dental product capable of temporarily controlling gingival bleeding, displacing marginal gingiva and drying the gingival sulci around a tooth.

The pastes allow the dental clinician to perform operative procedures without interference from blood, moisture or hyperplastic soft tissue.

Simplee Retraction Paste is supplied in:

In 0.24 gram single-patient use, unit dose capsules, designed for use with a compatible dispensing gun.

**INDICATIONS FOR USE**  
Simplee Retraction Paste is indicated for temporary retraction and hemostasis of the marginal gingival tissues during clinical procedures such as taking subgingival and per gingival dental impressions, seating of fixed prostheses, and restorative dentistry.

**CONTRAINDICATIONS FOR USE**  
Simplee Retraction Paste should not be swallowed, and is contraindicated for use on or by persons who are allergic to or sensitive to aluminum chloride, silicones or similar products. Symptoms may include dermatitis or inflammatory reactions. If this occurs, discontinue use and obtain medical care immediately. The product should not be used in the presence of, and is not a treatment for, active periodontal disease.

**PRECAUTIONS FOR USE**  
• Simplee Retraction Paste should be used in gingival sulci above the level of the junctional epithelium.  
• The tip of the unit dose capsule should be held at the mouth of the sulcus, without actually entering into it. The paste should be expressed slowly into the sulcus, until no residue remains in the gingival sulcus.  
• After the material has been in the sulcus for approximately 2 minutes, it must be completely removed. The sulcus should be thoroughly rinsed with water and inspected for residual material so none is left behind on the gingiva or the tooth surface.

**PRODUCT WARNINGS AND CLINICAL INFORMATION**  
• Store Simplee Retraction Paste in a cool dark place prior to use, sealed in the foil package.

• Unit dose capsules are single-patient use only, and should be discarded after use.

• Avoid sunlight or nearby heat sources. Shelf life will be reduced if not stored properly.

• Simplee Retraction Paste, like many agents used in dentistry to control bleeding, contains Aluminum Chloride. This may cause corrosion on some plastic or metal surfaces (even stainless steel). If left in extended contact, To protect dental instruments, always remember to rinse off Simplee Retraction Paste from their surface immediately after each use.

• If product accidentally contacts skin or eyes, wash with copious amounts of water. Obtain medical care immediately if necessary.

• Do not ingest this product, and keep it away from children.

**GENERAL INSTRUCTIONS FOR USE**  
1. After tooth preparation is complete, rinse the sulcus with water. Gently dry with compressed air at an angle to the open sulcus, not directly into it.

2. Open the resealable foil package by tearing off the top where indicated. Save the bag for later storage. Remove a unit dose capsule from the package.

3. By using a unit dose capsule, insert it properly into a compatible gun, remove the cap and "start" the flow by extruding a small amount onto a pad first. It is recommended that a barrier bag is used with the compatible gun.

4. Express Simplee Retraction Paste around the tooth into the mouth of the gingival sulcus, until it is slightly overfilled with material. Do not force the capsule into the tissues. Allow the material to remain undisturbed in the sulcus for 2 minutes.

5. To increase gingival retraction, pre-made cotton compression caps (or standard cotton rolls cut into short sections) may be placed over the tooth, followed by 2 minutes of biting pressure by the patient's opposing teeth.

6. Simplee Retraction Paste remains pliable, and does not fully dry out in the sulcus. After 2 minutes, rinse it away with water and air and remove

any remaining material using a non-cutting hand instrument or cotton pledge. Rinse and dry again. The sulcus is now ready for the impression or restorative procedure.

8. Unit dose capsules are for single-use on one patient only. Each time Simplee Retraction Paste is used on a different patient, apply a new unit dose capsule.

11. If you have any questions, call 1-800-243-7446 or e-mail info@parkell.com.

TIPS FOR DENTAL PRACTITIONERS

The following information should help dental practitioners reduce the risk of cross-contamination between patients when using multiple-use dental dispensers: apply disposable barrier sleeves/wraps over multiple-use dental dispensers; utilize multiple-use dental dispensers before use with each patient; use new, uncontaminated gloves when handling multiple-use dental dispensers;

utilize dental assistants to dispose material for the dentist; avoid contact of the reusable parts (e.g., the body of the multiple-use dental dispenser) with the patient's mouth; do not reuse the multiple-use dental dispenser if it becomes contaminated; do not reprocess a contaminated multiple-use dental dispenser by using chemical wipes or disinfectants; do not immerse multiple-use dental dispensers in a high level chemical disinfectant, as this may damage the dispenser and the material contained in the device; do not sterilize multiple-use dental dispensers, as this may damage the material contained in the device.

**DISPOSAL:**  
Dispose of the product in accordance with local regulations.

**EVENTS:**  
Reportable incidences should be reported to Parkell and the competent authority in which the user and/or patient is established.

**WARRANTY AND TERMS OF USE**  
For full warranty and Terms of Use information, please see [www.parkell.com](http://www.parkell.com). Safety Data Sheets (SDS) are available at [www.parkell.com](http://www.parkell.com). Parkell's Quality System is certified to ISO 13485.

**AVAILABLE PRODUCTS**  
Simplee Retractionpaste Capsules — (30) unit dose capsules [0.24g/capsule]

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