SIMPLEE®

All you need. No more. But no less.

SIMPLEE W+

GB DIRECTIONS FOR USE

SPECIFICATIONS

Item	Size	Taper	Max diameter	16mm	13mm	9mm	6mm	3mm
Small	20	.07	1.20	1.20	0.87	0.80	0.54	0.38
Primary	25	.07	1.20	1.20	0.97	0.82	0.61	0.44
Medium	35	.06	1.20	1.20	0.95	0.88	0.63	0.49
Large	45	.05	1.20	1.20	0.91	0.88	0.66	0.55

Model	Size	Speed(RPM)	Torque	Length
Small	20	350	3-4 N.cm	21mm, 25mm, 31mm
Primary	25	350	3-4 N.cm	21mm, 25mm, 31mm
Medium	35	350	3-4 N.cm	21mm, 25mm, 31mm
Large	45	350	3-4 N.cm	21mm, 25mm, 31mm

Simplee W+ Tunnel file				Length
Size	Taper	Speed (RPM)		
15	02	300-350		21mm, 25mm, 31mm

RAW MATERIAL

The whole cutting area of W+ system files are made of Nickel Titanium alloy.

SPECIFICATIONS

- Simplee W+ Tunnel file 15/02
- Simplee W+ Shaping files

Small W+	20/07
Primary W+	25/07
Medium W+	35/06
Large W+	45/05

FUNCTION

- Simplee W+ Tunnel files are used in the purpose of building a smooth tunnel during preparation by mechanically powered Simplee W+ files system.
- Simplee W+ Shaping files are used aiming at shaping and cleaning the root canal by mechanically powered Simplee W+ files system

WARNINGS

- Please confirm that patient has no allergy to Nickel Titanium alloy and other medical reagent before the treatment.
- These instruments are intended for use only in dental root canal treatment, users must follow a correct and reasonable way under medical environments.
- Please use a rubber dam system to ensure patients' safety when deliver endodontic treatment.
- Simplee W+ system are all disposable instruments, to avoid cross contamination and ensure it can reach a optimum performance, please abandon it after a single use.

PRECAUTIONS

- Safety equipment (gloves, glasses, rubber dam) is required for patients' and self-protection.
- These instruments can only be used after sterilization, please using these files under medical environment.
- Simplee W+ system are all heat-treated instruments and showing a greater flexibility performance than Simplee W system, it's a normal phenomenon that it might appear a little bent but it won't bring any disturbance for use.
- Reciprocating motion is recommended to match the Simplee W+ system.
- Watch out for the disposable warnings, in order to get a ideal performance of these instruments please not to try to use more than once.
- Please use Simplee W+ Tunnel file and Simplee W+ Shaping file in a appropriate case and in a correct way.
- Before using Simplee W+ Tunnel file, scouting by hand files is required and necessary.
- Remove debris out frequently from screwing-in problems
- Normally shaping the canal with EDTA for entering the canal smoothly and irrigate sufficiently after removing debris created by shaping the root canal.
- If you come across a situation that it's difficult to enter the deeper part of the canal, please stop and check both morphology of the canal and whether there are signs of deformation of files or not, then confirm the orifice is expanded wide enough for subsequent use of shaping files.

OPERATING STEPS

- Preoperative radiographic evaluation**
Observe the configuration of canal, then confirm the quantity, width, length, curvature, and choose a most suitable file system to deliver a treatment.
- Access**
Prepare a straight access to canal orifice.
- Scouting**
Explore the canal up to size #10 hand use file with EDTA.
- Smooth path done by Simplee W+ Tunnel file**
Create a smooth path and confirming the whole working length can be totally reached after the orifice is well expanded in order to be well prepared for subsequent use of shaping files, as a excellent connection between hand use files and rotary files.
- Shaping with Simplee W+ Shaping file**
On the basis of created glide path, shape the canal with Simplee W+ shaping file in the condition of EDTA. Normally choose PRIMATY file (25/07) as the initial file to start, changing the most optimal file subject to morphology of the canal.

PRODUCT STORAGE CONDITIONS

- Take it lightly and put it lightly during transportation and storage, do not stack it, to prevent damage to the packaging materials.
- Avoid storage in high temperature, high humidity and direct sunlight, and the products should be stored at room temperature.

WASTE TREATMENT

After use, follow good practice for dental root-canal instruments and must be placed in a solid container that should be used for collecting cutting or bonding instruments (e. g. needle or disposable scalpel).

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Artikel	Größe	Konizität	Max. Durchmesser	16mm	13mm	9mm	6mm	3mm
Small	20	.07	1.20	1.20	0.87	0.80	0.54	0.38
Primary	25	.07	1.20	1.20	0.97	0.82	0.61	0.44
Medium	35	.06	1.20	1.20	0.95	0.88	0.63	0.49
Large	45	.05	1.20	1.20	0.91	0.88	0.66	0.55

Model	Größe	Geschwindigkeit (RPM)	Drehmoment	Länge
Small	20	350	3-4 N.cm	21mm, 25mm, 31mm
Primary	25	350	3-4 N.cm	21mm, 25mm, 31mm
Medium	35	350	3-4 N.cm	21mm, 25mm, 31mm
Large	45	350	3-4 N.cm	21mm, 25mm, 31mm

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