Permaplast LH Hybrid Cavifils

Stand:07/1024

DE Produktbeschreibung

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REF 100 132 A 3,5

GB Product description

M+W Permaplast LH Hybrid Cavifils is a light-cured, radiopaque filling material for restorations of all cavity classes.

Indications

Loss of tooth substance due to caries, trauma, tooth abrasion or development disorders.

Contraindications

Do not use directly on the exposed pulp. Do not use the material where there are contact allergies or known allergies to any of the components.

Intended use

Restorations of all cavity classes.

Limitations of use

Application of M+W Permaplast LH Hybrid Cavifils is contraindicated if a dry working area or the recommended application technique are not possible.

Intended users

Dentist

Patient target group

Persons receiving treatment as part of a dental procedure.

Recommended use

- Clean the tooth prior to the preparation. Remove all cleaning residue with water spray. Determine the Vita shade. The use of a rubber dam is highly recommended.
- Prepare the cavity in accordance with the established rules for adhesive technique. Bevel the enamel margins. Ensure that the areas close to the pulp are protected.
- If necessary, etch using your preferred etching technique such as total-etch. Follow the instructions for use for the bonding agent used.
- Apply the bonding agent according to the instructions for use.
- If necessary, apply an underfilling of low-viscosity composite or compomer. We recommend using a matrix.
- Apply the desired shade of M+W Permaplast LH Hybrid Cavifils to the cavity with each layer having a maximum thickness of 2 mm. Polymerize each layer for 40 seconds with a light intensity of at least 600 mW/cm² or 20 seconds with a light intensity of at least 1000 mW/cm².

- Removal of any excess material, contouring, and finishing are done using diamond finishing tools and flexible polishing discs of decreasing grain size. M+W Permaplast LH Hybrid Cavifils can be given a high shine using polishing paste together with polishing cups or discs.

Notes for use

- If the material is applied to the patient's mouth with a single use applicator, the single use applicator must be used for this one patient only for hygienic reasons.
- Light-curing units should emit at 450 nm and should be checked regularly. The light intensity should be at least 600 mW/cm². Place the light-curing unit as close as possible to the filling material.
- Do not remove the oxygen-inhibited layer because this is required for the bond with the next layer.
- Follow the instructions for use for other materials that you use in conjunction with M+W Permaplast LH Hybrid Cavifils.

Interactions

- Materials containing eugenol, moisture and oily air may inhibit polymerization at the contact area. Avoid materials containing eugenol, moisture and air containing oil!

- The use of cationic oral rinses, plaque indicators and chlorhexidine may lead to discoloration.

Residual risks/side effects

There are no known side effects to date. The residual risk of hypersensitivity to components in the material cannot be ruled out.

Warnings/precautions

- Keep out of the reach of children!
- Avoid contact with the skin! In the event of accidental skin contact, immediately wash the affected area thoroughly with soap and water.
- Avoid contact with eyes! In the event of accidental contact with the eyes, immediately rinse thoroughly with plenty of water and consult a physician if necessary.
- Follow the manufacturer's instructions for other products that are used with the material/materials.
- Serious incidents involving this product must be reported to the manufacturer and to the responsible registration authorities.

Product performance characteristics

- The product complies with ISO 4049: 2019, Type 1, Class 2, Group 1.

- Polymerization depth ≥ 2 mm with a 20 second light-curing time when the light intensity of the lamp is ≥ 1000 mW/cm²
- Polymerization depth ≥ 2 mm with a 40 second light-curing time when the light intensity of the lamp is ≥ 600 mW/cm²

Storage/disposal

- Store at room temperature (15-25 °C / 59-77 °F). Close syringe / composite tip immediately after use so that the material does not cure in the ambient light.
- Do not use after the expiration date.

Composition

Dental glass, bis-GMA, TEDMA, UDMA, EBPADMA, SiO₂, additives. Inorganic filling materials: approx. 57 vol% (0.02 -- -1.5 µm).

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