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Stand: 15/019

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Seite 2-3

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Ⓒ INSTRUCTIONS FOR USE

Page 4-5

Page 14 TECHNICAL DATA

Ⓕ MODE D'EMPLOI

Page 6-7

Page 14 TECHNICAL DATA

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Pagina 8-9

Pagina 14 TECHNICAL DATA

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Strana 10-11

Strana 14 TECHNICAL DATA

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Oldal 12-13

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Stand: 15/10/19

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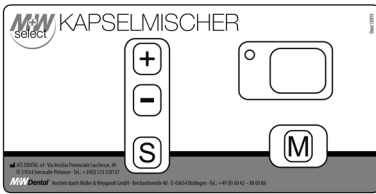
GB INSTRUCTIONS FOR USE

INTENDED USE

M+W Kapselmischer is a universal mixer for amalgam, glass ionomer cements and other pre-dosed dental materials in capsules and syringes. The apparatus has been developed solely for use in dentistry. Liability cannot be accepted for damage resulting from misuse or failure to observe the Operating Instructions. The user is responsible for testing the M+W Kapselmischer for its use and suitability for the intended purposes. The intended use also includes the observation of the notes and regulations in these Operating Instructions. The M+W Kapselmischer has been shipped from the manufacturer in a safe and technically sound condition. The apparatus is built according to: Directive 93/42/EEC and subsequent amendments contained in according with the Directive Rule 89/336/CEE, 92/31/CEE and 98/68/CEE and the following harmonized standards have been applied: EN 61010-1 - EN61010-2-81-EN61326-1 - EN61000-3-2 – EN6100-3-3 - ISO 7488 :1991

SAFETY

CAUTION !

Please read these instructions carefully before you set up and start the device. As with all technical equipment, this unit will function properly and be safe to operate only if both the general safety regulations and the specific safety instructions laid out in these Operating Instructions are followed.

1. The unit must be operated in accordance with the following instructions. We accept no liability for any damage arising from use of the unit for any other purpose.
2. Prior to setting-up the device, ensure that the voltage stated on the rating plate is identical to the available main voltage. Use of the device at different voltages may lead to the destruction of the unit.
3. The unit must be operated with the cover intact and closed, otherwise, there is a risk of injury.
4. The cover must not be opened until the capsule holder has come to a complete stop.
5. Only the capsule to be mixed but no other objects may be present in the space below the cover prior to initiating the mixing cycle.
6. Placement or operation of the unit on non-level surfaces is not permissible.
7. For operation of the unit, connect the main plug to an adequate electrical socket with grounded lead. If an extension cable is used, ensure that the grounded lead is not interrupted.
8. In order to prevent electrical shock hazards do not insert any objects into the unit with the exception of replacement parts properly installed in accordance with the present instructions.
9. The device should be opened and repairs be carried out only by qualified personnel.
10. Only adequately labelled original replacement parts may be used in the repair and maintenance of this unit. Any damage resulting from the use of third-party replacement parts voids the liability of M+W Dental.
11. Do not expose the device to solvents, flammable liquids, and strong heat sources as these may damage the plastic housing.

DENTAL MATERIALS USED

The corresponding instructions for use of the material suppliers have to be observed.

ASSUMPTION OF IMPAIRED SAFETY

If it has to be assumed that safe operation is no longer possible, the power must be disconnected to avoid accidental operation. This may be the case, for example, if the apparatus is visibly damaged or no longer works correctly. A complete disconnection from the power supply is only ensured when the power cord is disconnected from the power source.

REMOVAL OF TRANSIT SCREWS

BEFORE USING REMOVE TRANSIT SCREW FROM UNDERSIDE OF UNIT. (Fig A)

For safety during transit, the capsule mixer has been fitted with a transit screw prior to shipping. It is essential to remove this screw prior to using the capsule mixer.

- a) Place the unit on its side on a soft surface
- b) Using the supplied 3mm allen key, turn the transit screw anti-clockwise until it is loosened. Remove screws.
- c) Place the capsule mixer correct side up on a flat surface in an easily accessible location.

USE

Packaging part list: Electronic Control Console -Power Cord – Allen Key – DFU (Fig. D)

1. Connect the power cord with the socket at the rear of the apparatus and subsequently with the mains socket. (Fig. B)
2. Switch on the apparatus at the rear. (Fig. B)
3. Raise the safety cover and place the mixing capsule in the mixing fork (Fig. C)
 - Insert the capsule in the right cap of plastic fork (Fig. C-1)
 - Open the fork moving the left part (max. 8mm) (Fig. C-2)
 - Turn the capsule inside the plastic fork (Fig. C-3)
 - Check the capsule is clamp correctly.

M+W Kapselmischer allows two different techniques of working: manual or with memorized times.

Manual Working

1. Plan the wished time with the bottoms "+" and "-".
2. Push the bottom "S" (Start). On display you will see the decreasing time. When the operation is finished, it will remain planed.

Working with memorized times.

The machine is supplied with 10 planed times of mixing:

1. Push the button "M". It will switch on the yellow light which indicates you are working with memories and it will show a number with one figure which indicates the number of memory.
 2. Push the bottom "M" another time. It will show a number with two figures (only for two seconds) which is relative to mixing time corresponding to previous memory.
 3. Go on pushing the same bottom "M". It will show in succession the next number of memory with the relative time of mixing.
 4. When you have selected the right time of mixing, push the bottom "S" to make operation. During the mixing, you can see decreasing time. At the end of cycle, display will show the number of memory you used.
- If the safety cover is opened during the mixing process, the circuit is broken and the motor stops automatically. The mixing process might be finished by opening the safety cover or by pressing the button "S" again. When you turn off the main switch, the amalgamator remains planed on the last operation it executed. M+W Kapselmischer fits capsules between 30 and 36 mm of length and with 14/18 mm of diameter.

CLEANING

Switch off the apparatus by disconnecting the mains cord. Clean housing, keypad and safety cover with a soft cloth and customary aldehyde-free disinfecting solution. Do not clean with highly aggressive disinfecting solutions (e.g. solutions based on orange oil or with an ethanol content of more than 50%), solvents (e.g. acetone), or pointed instruments, which may damage or scratch the plastic. Make sure that no disinfectant, other liquid or foreign object enters the apparatus.

FUSE REPLACEMENT

Your device is protected by one fuse. The replacement procedure is as follows: **(Fig B1)**

- Switch off the unit and disconnect it from the main power supply.
- Insert a screwdriver in the slot of the fuse holder and open the flap.
- Remove the fuse
- Replace the fuse with one of the same type and value (1x T2.0A 250V).
- Close the flap.

Note: All the information necessary for the repair of faulty components is made available by the manufacturer to the technical personnel of the approved dealer network who carry out the work.

TROUBLE SHOOTING

1. Loud noise heard when trituration begins: remove transit screw.
2. Capsules are not secure: if the arms have become stretched, replace forks. (Contact an authorized service personnel for replacement/ installation)
3. Electrical or mechanical malfunction: Call your authorized service personnel.

WARRANTY PERIOD / WARRANTY CONDITIONS

M+W Dental extends a 2 year warranty against defects in materials or workmanship to the original purchaser of this product. The Manufacturer agrees to correct any defects which develop within the warranty period, either by repair or replacement, at its option. This warranty is valid providing factory inspection indicates that any such defect developed during normal and proper use subject to the conditions below.

- Claims for damage in shipment should be filed promptly with the transportation company.
- All shipments claimed defective can only be returned to the Manufacturer with the written consent of the Manufacturer.
- All returned products must be accompanied by a full description of discrepancy or malfunction.
- Only authorized service personnel are allowed to carry out repairs to the unit.
- The Manufacturer shall be released from all obligations under this warranty if repairs or modifications are made by persons other than the Manufacturer's own or authorized service personnel.
- Use only original parts exclusively to replace defective components.
- The product warranty does not cover any damage resulting from the use of third-party replacements parts.
- The Manufacturer shall be released from all obligations under this warranty in the event of improper installation; damage due to casualty; or obvious misuse including but not limited to damage from liquids.
- The Manufacturer shall in no event be liable for any consequential damages arising from the misuse of the product. Only the above stated warranty shall be applicable, and all other warranties, expressed or implied, are hereby excluded.

All correspondence concerning the product should specify the model and serial number.

LIMITATION OF LIABILITY

Except where prohibited by law, M+W Dental will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

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
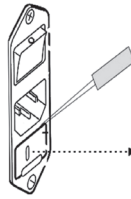
FIG „C“

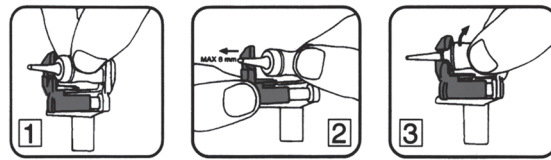TECHNICAL DATA

Manufacturer	M+W Dental
Model	M+W Kapselmischer
Dimensions	300 x 220 x 170
Materials	Metal
Noise	< 65 dBA
Supply Voltage	AC230V
Frequency	50/60 Hz
Nominal Power	110 VA
Class MD	I rule 12 in Annex IX 93/42/EEC
Console	IP21
7488 Classification	Type 1
Operating Mode	30" ON – 180" OFF
Max Working Time	30'
Fuses	1xT2.0A 250V

Conditions of use: Temp +5° - +40°

Shipping and Storage conditions:
Temp +5°C/+65°C (+41°F/+149°F) RH < 20-95% non-condensing

SYMBOL USED

	EEC 93/42		Supply Voltage
	Manufacturer		Model - Code
	Intermittent Operations		DFU available at www.mwdental.de
	Fuse		Ground
	This symbol remind to consult the present Operating and Maintenance Instruction Manual.		
	This symbol on the product or on its packaging indicates that this product may not be treated as household waste. Instead it shall be handled over the applicable collection point for the recycling of electrical and electronic equipment. By ensuring this product is disposed correctly, you will help prevent potential negative consequences for the environment and human health, which could otherwise be caused by inappropriate waste handling of this product. For more detailed information about recycling of this product, please contact your local city office, your household waste disposal service or the shop where you purchased the product.		
	High Voltage Inside. This equipment operates from A.C mains. Only authorized personnel to open unit.		

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