making work easy

- de
- en
- fr
- it
- es
- pt
- tr
- ru
- pl
- zh
- ja
- ko
- ar

ORTHO guide Set MT3

Made in Germany

21-9294 24112023

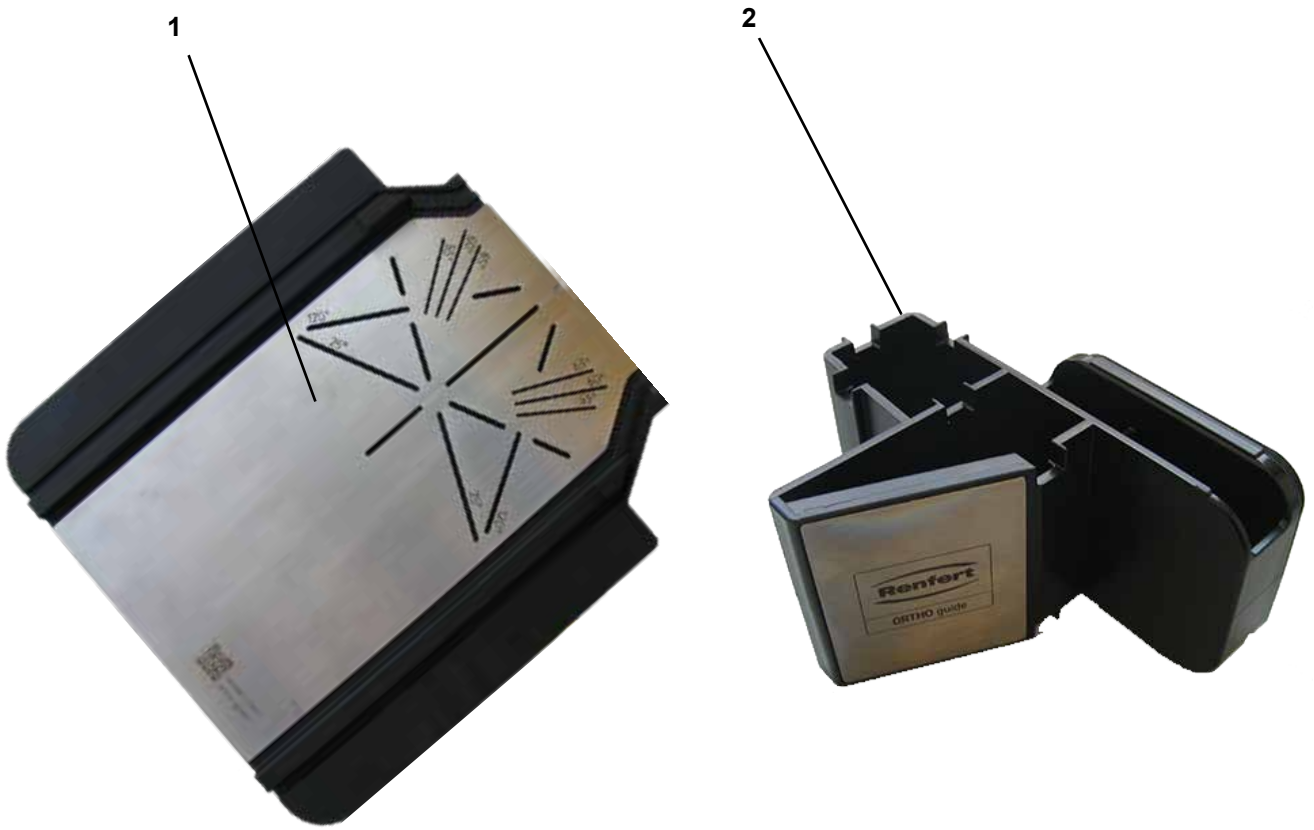
Fig. 1

1 Einleitung

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1 Introduction

These instructions describe the assembly and operation of the ORTHO guide set MT3.

1.1 Symbols used

Symbols with the following meanings are used in these instructions or on the equipment:

Danger

Immediate danger of injury. Consult accompanying documents!

Electrical voltage

Danger due to electrical voltage.

Caution

Risk of damage to the equipment if the instruction is not followed.

Notice

Provides the operator with useful information that eases use.

2 Safety

2.1 Intended use

The ORTHO guide set MT3 can be used to extend the MT3 trimmer with an orthodontic trimmer table.

2.2 Ambient conditions for safe operation

The ambient conditions for the trimmer shall apply.

2.3 Hazard and warning information

2.3.1 General information

- ▶ **If the unit is not used in accordance with the supplied operating instructions, the intended safety can no longer be guaranteed.**
- ▶ **Disconnect the unit from the mains power supply before carrying out work on the electrical parts.**
- ▶ **Defective equipment must be taken out of service immediately. Disconnect the power plug and secure the equipment against further use. Send the equipment in for repair!**
- ▶ **Information on REACH and SVHC is available on our website at www.renfert.com, in the Support area.**

2.4 Authorized persons

Operation and maintenance of the equipment may only be performed by trained personnel.

Any repairs not described in these operating instructions may only be performed by a qualified electrician.

2.5 Disclaimer

Renfert GmbH shall be absolved from all claims for damages or warranty if:

- ▶ **The product is employed for any purposes other than those specified in the operating instructions.**
- ▶ **The product is altered in any way other than those alterations described in the operating instructions.**
- ▶ **The product is not repaired by an authorized dealer or if original Renfert spare parts are not used.**
- ▶ **The product continues to be used despite obvious safety defects or damage.**
- ▶ **The product is subjected to mechanical impact or is dropped.**

3 Product description

en

3.1 General description

You can extend the MT3 model trimmer using the ORTHO guide set MT3.

The orthodontic model table and template enable precise trimming of the planning and diagnostic models.

3.2 Assemblies and functional elements

See Fig. 1 (on inside cover)

- 1 Orthodontic model table
- 2 Template

3.3 Scope of delivery

- 1 Orthodontic model table
- 1 Template
- 1 STEP-BY-STEP instructions
- 1 Operating Instructions
Demonstration models, 1 x upper jaw, 1 x lower jaw

4 Startup

4.1 Assembly – orthodontic model table

- ⇒ Pull the attached model table upwards from the ball head at the front edge and remove it.
- ⇒ Turn the adjustment fork to the 90° position with the long side facing upwards (see also the operating instructions of the trimmer).
- ⇒ Insert the orthodontic model table and press it onto the ball head again.
- ⇒ Attach the template.

The black surface of the orthodontic model table shows milling marks caused by the template after a short time. The marks do not affect the functionality of the unit.

5 Operation

The handling of the orthodontic model table and the template can be found in the enclosed step-by-step instructions.

There you will also find a link to user videos.

6 Cleaning / maintenance

6.1 Orthodontic model table and template

The orthodontic model table can be cleaned in the same way as the standard trimmer model table.

- ⇒ Remove the template.
- ⇒ Pull the model table upwards from the ball head at the front edge.
- ⇒ Clean the model table and the template using a brush as appropriate under running water.

Do not use any objects with sharp edges or solvent-based / abrasive cleaning agents. Never use cleaning agents containing solvents or strong acids (e.g. descaling agents).

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Renfert GmbH • Untere Gießwiesen 2 • 78247 Hilzingen/Germany