

SLIM-POLISHER A sind weiche, elastische Polierscheiben, ideal zum Beschleifen und Glätten von flexiblen Kunststoffen, Silikonem oder anderen weichbleibenden Materialien. Sie eignen sich auch zum Vorpolieren von Metall und anderen Kunststoffen. Die Scheiben passen sich flexibel den Konturen der zu bearbeitenden Oberflächen an. Sie zeichnen sich durch geringe Wärmeentwicklung aus, schmieren nicht und gewährleisten

durch ständig frei werdende Schleifpartikel gleichbleibende Ergebnisse.

**Dentaler Bereich:**

Weichbleibende Unterfütterungen, Mundschutz, Positioner usw.

SLIM-POLISHER A hat eine halboffene Struktur und ermöglicht mit dieser Körnung sowohl Abtragen, Abrunden und Glätten. Idealer Drehzahlbereich 4.000-12.000 U/Min. Schutzbrille tragen!




SLIM-POLISHER A

Lieferform:

10 SLIM-POLISHER A

Polierscheiben, Ø 28 mm, unmontiert, 6 Stützscheiben.

Korn: fein, orange

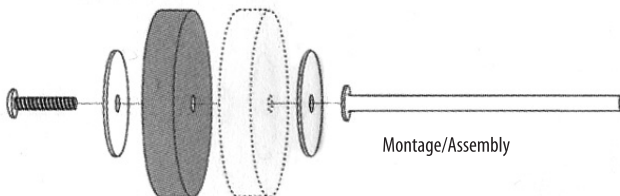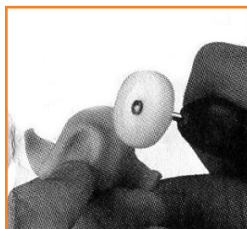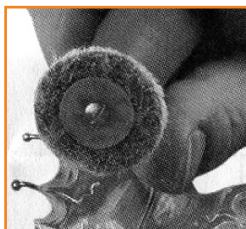

Montage/Assembly

**(GB) Directions for use**

SLIM-POLISHER A are soft, elastic polishing discs, excellently suited for grinding and smoothing flexible acrylics, silicones or other soft materials. They are also suited for prepolishing metal and other acrylics. The discs adapt flexibly to the contours of the surface which has to be worked. They distinguish by only small heat development, do not smear and en-

sure constant results due to particles which are polished off steadily.

**Dental field:**

Soft relines, mouthguards, positioners etc.

SLIM-POLISHER A has a half-open structure and due to this grain size it makes possible removal, rounding off and smoothing. Ideal speed range

4.000-12.000 revs./min. Wear protective glasses!

SLIM-POLISHER A

Supplies:

10 SLIM-POLISHER A polishing discs, Ø 28 mm, disassembled, 6 supporting discs

Grain size: fine, orange

**Letzte Überarbeitung/latest revision: 28.03.2017**

Stand:280318